

- Place the coverslip on a slide. If desired, iodine may be added to the wet mount for better detail. The edges may be sealed with a petroleum jelly/paraffin mixture to prevent drying of the specimen.

*Alternate Method - directly after step 6, without removing the tube from the centrifuge, remove 1 or 2 drops of the surface film with a Pasteur pipette or a freshly flamed wire loop. Do not use the loop as a "dipper", simply touch the surface with the loop portion. Proceed with examination.

PRECAUTIONS

Avoid contact of fixative solutions with the skin and eyes. Should contact occur, call a physician immediately. Flush with plenty of water. DO NOT DRINK. If ingested, call a physician immediately.

Occupational Safety and Health Act regulations (including Universal Precautions) should be used for handling all specimens.

If gelled, the fixative may be liquefied by placing in a 50°C water bath until clear and fluid.

For assistance please call our Technical Service Department toll-free at 1-800-528-0494 between the hours of 8 A.M. and 5 P.M. Eastern Standard Time.

CAS Numbers

10% Formalin		Zn-PVA	
Formaldehyde	50-00-0	Zinc Sulfate	7733-02-0
DI Water	7732-18-5	Ethanol	64-17-5
Na ₂ HPO ₄	7558-79-4	Acetic Acid	64-19-7
NaH ₂ PO ₄	7558-80-7	Glycerine	56-81-5
		PVA	9002-89-5
		Methanol	67-56-1
		DI Water	7732-18-5

BIBLIOGRAPHY

- ASMT. Recommended Procedures for the Examination of Clinical Specimens Submitted for the Diagnosis of Parasitic Infections. Am J Med Technol, 1978; 44:1101-1106.
- Brook MM, Goldman M. Polyvinyl alcohol fixative as a preservative and adhesive for protozoa in dysenteric stools and other liquid materials. J Lab and Clin Med 1944;34:1554-1560.
- Brooke MM, Norman C. The effectiveness of the PVA fixative technique in revealing intestinal amebae in diagnostic cultures. Am J Trop Med Hyg 1955;4:479-482.
- Burrows RB. Microscopic Diagnosis of the Parasites of Man, New Haven: Yale University Press, 1965.
- Erdman, Dean. Clinical comparison of ethyl acetate and diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol 1981;14:483-485.
- Garcia MS, Bruckner D. Diagnostic Medical Parasitology, 3rd ed. Washington D.C.: ASM Press, 1997: 594-822.
- Horen, WP. Modification of Schaudin's fixative. J Clin Microbiol 1981;13:204-205.
- Melvin DM, Brooke MM. Laboratory Procedures for the Diagnosis of Intestinal Parasites. Atlanta: U.S.H.H.S.(CDC), 1982;82:8262.
- Scholten TH, Yang J. Evaluation of unpreserved and preserved stools for the detection and identification of intestinal parasites. Am J Clin Path 1974;62:563-567.
- Young, Kirk H, et al. Ethyl acetate as a substitute for diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol 1979;10:852-853.

Protocol Zn-PVA Vials

Protocol	Item #
Protocol Zn-P VA 20/pk	23-005-37
Protocol Zn-PVA/ Formalin Kit, 2x10 v/pk	23-005-45
Protocol Zn-PVA/ Formalin/Clean, 3x10 v/pk	23-005-28
Protocol Zn-PVA/ Formalin/C&S, 3x10 v/pk	23-005-32

Other Protocol Parasitology Products

Protocol	Item #
Protocol LV-PVA Vials, 20/pk	23-005-29
Protocol SAF Vials, 20/pk	23-005-41
Protocol Clean Vials, 20/pk	23-005-31
Protocol 10% Buff. Formalin Vials, 20/pk	23-005-46
Protocol Modified (Cu) PVA Vials, 20/pk	23-005-33
Protocol C & S Vials, 20/pk	23-005-43
Protocol MIF Vials, 20/pk	23-005-35
Protocol Cary Blair Vials, 20/pk	23-005-47
Protocol LV-PVA / Formalin Kit, 2x10 v/pk	23-005-23
Protocol SAF / Clean Kit, 2x10 v/pk	23-005-44
Protocol LV-PVA / Clean Kit, 2x10 v/pk	23-005-25
Protocol SAF / C&S Kit, 2x10 v/pk	23-005-39
Protocol Modified (Cu) PVA/Formalin, 2x10 v/pk	23-005-27
Protocol PVA/ Formalin/Clean, 3x10 v/pk	23-005-22
Protocol PVA/Formalin/C&S, 3x10 v/pk	23-005-24
Protocol SAF/Clean/C&S, 3x10 v/pk	23-005-26
Protocol Mod. (Cu) PVA/Formalin/Clean 3x10 v/pk	23-005-30
Protocol MIF Kit	23-005-34
Protocol MIF Bulk Kit	23-005-36
Protocol Trichrome, 500 mL	23-005-38
Protocol LV-PVA Bulk, 500 mL	23-005-49
Protocol Zn-PVA Bulk, 500 mL	23-005-40
Protocol Bulk Tubes and caps, 15 mL	23-005-42
Protocol Iodine, 50 mL	23-005-48
Protocol 50 mL Concentration System, 120 ea.	23-005-50
Protocol 15 mL Concentration System, 50 ea.	23-005-51
Protocol 15 mL Concentration System, 450 ea.	23-005-52
Protocol Ethyl Acetate	23-005-68

840560 (R0)

Protocol™

Parasitology System

Application for Zn-PVA and Formalin Vial Systems

PURPOSE:

Protocol Parasitology Systems provide standardized procedures for the routine collection, transportation, preservation and examination of stool specimens for intestinal parasites. Protocol Systems are for use by patient or healthcare worker and help minimize the need for immediate transport and processing of specimens.

EXPLANATION

Diagnosis of enteric parasitic disease is confirmed by isolation and identification of pathogenic organisms in fresh stool specimens. Procedures such as freezing, incubation and refrigeration do not insure recovery and identification of all intestinal parasites. The two vial method, PVA & 10% formalin, has been demonstrated as an effective transport and fixative technique (2,3,4,6). Proper use of the Protocol Parasitology System assures the parasitologist that diagnostic stages of intestinal parasites, if present, will be preserved. Zn-PVA acts as a preservative/fixative for protozoan trophozoites. Slides may be stained using trichrome or hematoxylin. Formalin is an all-purpose fixative that is appropriate for helminth eggs and larvae and for protozoan cysts. The buffering system of Protocol formalin helps maintain organism morphology. Formalin preserved specimens may be examined directly or concentrated for recovery of eggs, larvae and protozoan cysts.

CONTENTS:

Protocol ZN-PVA Vials (23-005-37) contain twenty 15 mL vials of ZN-PVA fixative. ZN-PVA/Formalin Kits (23-005-45) contain ten kits each consisting of one 15 mL vial of ZN-PVA fixative and one 15 mL vial of 10% formalin. Protocol ZN-PVA/Formalin/Clean Kits (23-005-28) contain ten kits each consisting of one 15 mL vial of ZN-PVA fixative, one 15 mL vial of 10% formalin and one Clean vial. Protocol ZN-PVA/Formalin/C&S Kits (23-005-32) contain ten kits each consisting of one 15 mL vial of ZN-PVA fixative, one 15 mL vial of 10% formalin and one 15mL vial of C&S medium. Each vial has a built-in collection spoon. Directions for use are also included.

MATERIALS NOT PROVIDED

Ethyl acetate
Zinc sulfate solution
Physiological saline
Cotton tipped applicator sticks
Microscope slides and coverslips
Centrifuge
Microscope
Transfer pipettes

SPECIMEN COLLECTION

1. Caution: Patient should not use antacids, barium, bismuth, antibiotics, anti-malarial agents, antidiarrheal medication or oily laxatives prior to specimen collection. After administration of any of these compounds, specimen collection should be delayed for 5 to 10 days or at least two weeks after barium or antibiotics.
2. Several specimens, collected intermittently over several days, should be examined to insure recovery of organisms.
3. Specimens must be collected properly to avoid contamination with urine or water (see collection instructions). Specimens are best collected in a bedpan. A clean dry container such as a milk container may be used by removing the top and washing thoroughly. Another option is to place plastic wrap over the toilet seat opening.
4. A suitable area (i.e. bloody, slimy, watery) from the sides, ends and middle of the stool should be selected using the collection spoon provided. Fill with sufficient stool to bring the liquid level up to the "Fill" line. This will result in approximately 5 mL of sample.
5. Stir each specimen with the spoon provided, tighten the cap and shake firmly until the specimen is adequately mixed. When mixing is complete the specimen should appear uniform.
6. Complete the label on each vial and replace the vials in the plastic bag.

SPECIMEN PROCESSING

The Protocol System allows for a variety of procedures to be utilized. A complete examination should include at least four steps: gross examination, direct microscopic examination, slide staining and one or more concentration procedures. While each laboratory should follow its own established technique, the following gives directions for commonly accepted procedures:

1. Gross examination: record the presence of blood, worms, mucus or proglottids. If the consistency of the stool can be determined, it may give an indication of the types of organisms present and should be recorded.
2. Direct microscopic examination of formalin preserved specimen:
 - a. Place a clean glass slide on a sheet of newsprint. Add a drop of 0.85% saline (iodine may be substituted) to the slide.
 - b. Add a representative sample of formalin fixed specimen to the drop of saline and mix thoroughly. The newsprint should be just legible through the slide.

- c. Place a wide coverslip on the suspension and examine immediately.
 - d. For a temporary seal, a cotton-tipped applicator stick dipped in equal parts of heated paraffin and petroleum jelly should be used.
3. Permanent slides for staining with trichrome stain or iron-hematoxylin:
 - a. Allow specimen to fix for at least 30 minutes in Zn-PVA. Mix thoroughly with two applicator sticks. Pour a small amount of the Zn-PVA fixed material onto a paper towel and allow to stand for three minutes. (Allow for absorption of the excess PVA.) Do not eliminate this step.
 - b. Using an applicator stick, apply (Do Not Smear) some of the stool material from the paper towel onto one or more clean glass slides to the edge of the slides.
 - c. Dry the slides overnight at room temperature or for several hours in a slide warmer or 37°C incubator. Accelerated drying is not recommended and may cause distortion of the specimen morphology. Do not proceed until the slides are completely dry.
 - d. Once the slides have dried, proceed with staining (Refer to respective stain insert sheet).

NOTE: Slides made from very watery specimens may require an additional time to dry completely.

4. Concentration procedures: One or more concentration procedures should be employed. There are two types of concentration procedures, sedimentation and flotation, both of which are designed to separate protozoan organisms and helminth eggs and larvae from fecal debris by centrifugation and/or differences in specific gravity. Each procedure may vary in respect to the efficacy with which it recovers specific organisms. No one concentration procedure works equally well for all parasites and laboratory personnel should consult available literature for additional information. The following are two common procedures:

A. Formalin-ether/ethyl acetate sedimentation:

1. Mix the formalin or Zn-PVA fixed specimen thoroughly. The specimen is now ready for processing with the Protocol Stool Concentration System. See the appropriate package insert for further directions.
2. If the Protocol Stool Concentration System is not available, strain a sufficient quantity through a layer of wet mesh gauze to proceed with step 3. This will vary with the size and density of the specimen.
3. Add saline (or 10% formalin) almost to the top of the tube, mix completely and centrifuge at 500 x g for 10 minutes (1800-2200 rpm). If the resulting precipitate is not 0.5 - 1 mL, resuspend, add or remove specimen and recentrifuge.
4. Decant the supernatant fluid. A second wash may be used if necessary.
5. Add approx. 10 mL of 10% formalin (fill the tube half full only), resuspend sediment mix and allow to stand for five minutes. If the amount of sediment left in the bottom of the tube is very small or the original specimen contained a lot of mucus, do not add ethyl acetate in step 8; merely add the 10% formalin, spin, decant, and examine the remaining sediment.

6. Add 3-5 mL of ethyl acetate or ether, tighten the cap and shake vigorously for at least 30 seconds.
7. Centrifuge at 500 x g for 10 minutes (1800-2200 rpm).
8. The resulting solution will separate into four layers:
 - a) Top layer: ethyl acetate or ether
 - b) Second layer: plug of debris
 - c) Third layer: formalin
 - d) Bottom layer: sediment
9. After ringing the plug of debris from the side of the tube with an applicator stick, carefully decant the top three layers. While keeping the tube inverted, a cotton swab may be used to remove debris sticking to the sides of the tube. This is particularly important for obtaining suitable results with ethyl acetate and prevents solvent bubbles in the wet mount.
10. Add a few drops of physiological saline or 10% formalin to resuspend the remaining sediment. Add a small amount of material to the slide. If the resulting slides are too dense (newsprint should be legible through them) more saline or 10% formalin may be added. Add a coverslip.
11. Examine and record results. As with the direct wet mount, iodine can be added to enhance morphological detail.

B. Zinc sulfate flotation

1. Thoroughly mix a representative portion of the formalin fixed stool suspension or fresh specimen in a 15 mL centrifuge tube and fill with tap water to approximately 10 mL. The specimen amount will vary according to the size and density.
2. Centrifuge at 500 x g for 10 minutes (1800-2200 rpm).
3. Sediment should be 0.5 - 1 mL. If needed, adjust the suspension by adding formalin fixed material or diluting with water. Once sediment is at the desired 0.5 - 1 mL amount, decant the supernatant. Repeat the wash if necessary.
4. Fill the tube approx. half full with zinc sulfate solution and resuspend the sediment by mixing thoroughly with applicator sticks. (Zinc sulfate solution must have a specific gravity of 1.2. Adjust as needed.)
5. Add additional zinc sulfate solution to within one inch of the top.
6. Centrifuge at 500 x g for 10 minutes (1800-2200 rpm).
7. Carefully remove the tube and place upright in a test tube rack without disturbing the contents. There should be two resulting layers: a small amount of sediment in the bottom of the tube and a layer of zinc sulfate.
8. Without overflowing the tube, fill with additional zinc sulfate to the top of the tube.
9. Place a clean coverslip on top of the tube. If the coverslip does not make contact with the solution in the tube, carefully add more liquid.
10. Do not disturb the tube or coverslip for fifteen minutes.
11. With a swift motion, lift the coverslip straight upward so that a drop of liquid is contained on the coverslip. This drop will contain cysts or eggs if they are present in the specimen.

Thermo
SCIENTIFIC

Fisher Diagnostics
a division of Fisher Scientific Company, LLC
a subsidiary of Thermo Fisher Scientific Inc.
Middleton, WI 53562-1055 USA
Phone: 1-800-638-6641 / 1-540-408-2200
Fax: 1-540-408-4122