## Wilkins Chalgren Anaerobe Broth (Dehydrated)

EN

**REF** CM0643B & CM0643R

### Intended Use

Wilkins Chalgren Anaerobe Agar (CM0643B/R) is a medium for the growth of anaerobes and recommended for antimicrobial susceptibility testing.

The devices are for professional use only, are not automated, nor are they companion diagnostics.

### Summary and Explanation

Wilkins-Chalgren Anaerobe Broth supports the luxuriant growth of anaerobic organisms. It is derived from Wilkins Chalgren Agar CM0619B. Where studies on antimicrobial susceptibilities are to be made in both broth and agar, this medium is found to be of particular value.

### Principle of Method

Wilkins Chalgren Anaerobe Broth includes yeast extract to supply vitamins and other growth factors such as purines and pyrimidines, that are necessary for good growth of *Peptostreptococcus anaerobius* and *Bacteroides melaninogenicus*. Arginine is added to ensure sufficient amino-acid is available for the growth of *Eubacterium lentum*. Pyruvate is present as an energy source for asaccharolytic cocci such as *Veillonella*. It also acts similarly to catalase and degrades traces of hydrogen peroxide, which may be produced by the action of molecular oxygen on medium constituents and interfere with the metabolism of anaerobes. Haemin is found to be essential for the growth of *Bacteroides* species and menadione for *Bacteroides melaninogenicus*.

Peptones derived from the single protein sources casein and gelatin are used to improve standardisation of the medium. The early heavy growth that is usual may reflect the absence of Eh-reducing substances that can be inhibitory to some organisms.

### Typical Formula

	<u>grams per litre</u>
Tryptone	10.0
Gelatine Peptone	10.0
Yeast Extract	5.0
Glucose	1.0
Sodium Chloride	5.0
L-Arginine	1.0
Sodium Pyruvate	1.0
Menadione	0.0005
Haemin	0.005

### Materials Provided

CM0643B: 500g of Wilkins Chalgren Anaerobe Broth.  
CM0643R: 2.5kg of Wilkins Chalgren Anaerobe Broth.

500g of dehydrated Wilkins Chalgren Anaerobe Broth yields approximately 15.2L after reconstitution.

### Materials Required but Not Supplied

- Inoculating loops, swabs, collection containers.
- Incubators.
- Quality control organisms.
- Suitable final containers.

### Storage

- Store product in its original packaging between 10°C and 30°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Protect from moisture.
- Store away from light.
- Allow reconstituted product to equilibrate to room temperature before use.
- 

Once reconstituted, store media between 2°C and 8°C and keep away from light.

### Warnings and Precautions

- For in vitro diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.

- Do not use the product if there is any visible damage to the packaging (pot or cap).
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products.
- Ensure the lid of the container is kept tightly closed after first opening and between use to minimise moisture ingress, which may result in incorrect product performance.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product ([www.thermofisher.com](http://www.thermofisher.com)).

### Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

### Specimen Collection, Handling and Storage

Specimen should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI).

### Procedure

Suspend 33g in 1 litre of warm distilled water. Mix well, distribute into final containers and sterilise by autoclaving at 121°C for 15 minutes.

Inoculate the specimen into the medium. Incubate the broth for 24 - 48 hours at 37°C under anaerobic conditions. Visually inspect broths to assess growth under good lighting.

### Interpretation

Visible turbidity in Wilkin Chalgren Anaerobe Broth after incubation signifies growth.

### Quality Control

It is the responsibility of the user to perform Quality Control testing taking into account the intended use of the medium, and in accordance with any local applicable regulations (frequency, number of strains, incubation temperature etc.).

The performance of this medium can be verified by testing the following reference strains.

Incubation Conditions: 24 – 48 hours @ 37°C, anaerobic.

<b>Positive Controls</b>	
Inoculum level: 10 – 100 cfu (colony forming units) A satisfactory result is represented by visible growth.	
<i>Clostridium perfringens</i> ATCC® 13124™	Turbid growth
Inoculum level: 1E+04 – 1E+06 cfu A satisfactory result is represented by visible growth.	
<i>Prevotella loescheii</i> ATCC® 15930™	Turbid growth
<i>Clostridium tetani</i> ATCC® 9441™	Turbid growth
<i>Clostridium sporogenes</i> ATCC® 19404™	Turbid growth
<i>Clostridium perfringens</i> NCTC 10613	Turbid growth
<i>Clostridium sphenoides</i> ATCC® 19403™	Turbid growth
<i>Bacteroides fragilis</i> ATCC® 25285™	Turbid growth
<i>Peptostreptococcus anaerobius</i> ATCC® 27337™	Turbid growth
<i>Fusobacterium nucleatum</i> ATCC® 27337™	Turbid growth
<i>Streptococcus pyogenes</i> ATCC® 19615™	Turbid growth

<i>Eggerthella lenta</i> ATCC® 43055™	Turbid growth
<i>Bacteroides thetaiotaomicron</i> ATCC® 29741™	Turbid growth

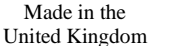
### Limitations

Identifications are presumptive and colonies should be confirmed using appropriate methods.

### Performance Characteristics

Accuracy has been demonstrated through review of internal QC data. Testing of well-characterised isolates in the QC processes is performed as part of the manufacture of each batch of the devices.

### Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Consult instructions for use or consult electronic instructions for use
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
	Made in the United Kingdom

ATCC Licensed  
Derivative<sup>®</sup>

©2022 Thermo Fisher Scientific Inc. All rights reserved.

ATCC and ATCC catalogue marks are a trademark of American Type Culture Collection.  
NCTC and NCTC catalogue marks are a trademark of National Collection of Type Cultures.  
All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, UK

For technical assistance please contact your local distributor.

**Revision information**

Version	Date of modifications introduced
2.0	2024-10-11 Minor formatting changes

## Wilkins Chalgren Anaerobe Broth (Dehydrated)

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©2022 Thermo Fisher Scientific Inc. Toate drepturile rezervate.
Toate celelalte mărci comerciale aparțin Thermo Fisher Scientific Inc. și subsidiarelor acesteia.